DuoEK™ S

Hand-held EKG Tracker

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User Manual

Contents

1. The Basics.....	2
1.1 Safety.....	2
2. Introduction.....	3
2.1 Overview.....	3
2.2 Unpacking.....	4
2.3 Symbols.....	5
3. Using the Monitor.....	6
3.1 Before Use.....	6
3.2 Connecting to the App.....	7
3.3 Taking EKG Recordings.....	9
3.4 Ending EKG Recordings.....	11
3.5 Real-time Monitoring.....	11
3.6 Taking Measurements without the App.....	12
3.7 Reviewing History Data.....	13
3.8 Setting the Heartbeat Sound.....	16
3.9 Factory Reset.....	17
4. Maintenance.....	17
5. Troubleshooting.....	18
6. Specifications.....	19

1. The Basics

Disclaimer

The EKG tracker is not a medical device and should not be used for any medical purposes or any medical conditions. It is intended only for general wellness use.

Only use the monitor as described in this instruction manual. Any other use is deemed improper and may result in damage to property or even cause personal injury.

The manufacturer or vendor cannot be held liable for damages incurred through improper or incorrect use.

1.1 Safety

Warnings and Advice

- Before using this device, please read this manual carefully and fully understand the warnings and risks.
- This device is not intended to replace medical diagnoses from a professional doctor.
- This device is not suitable for people wearing a pacemaker.
- Do not swim with or submerge the device in the water or other liquids.
- Do not cause the device to collide or crush it. If the casing is broken, stop using it.
- Keep this device away from children or pets.


- This device should not be used on people with sensitive skin or allergies.
- This device should not be placed in the following environments: direct sunlight, high temperatures, high humidity, or sources of flame.
- Users should inspect loose electrodes that can degrade performance or cause other problems.
- Do not use the device in a combustible environment.

2. Introduction

DuoEK S is a hand-held EKG monitor for measuring and recording EKG.

The free ViHealth app allows you to review the data and monitor EKG in real-time.

2.1 Overview

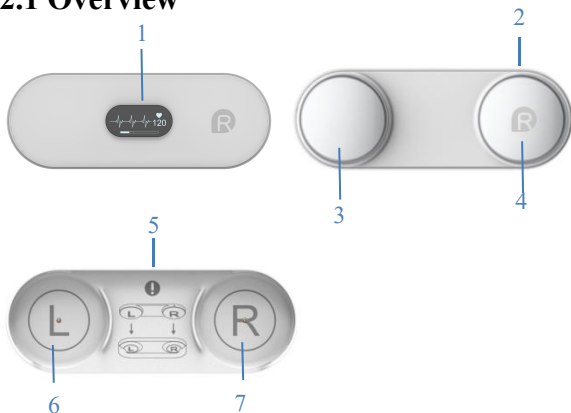1. Display screen

2. LED indicator

- **OFF:** The device is inactivated, or the battery has run out.
- **Green On:** Fully charged
- **Green Slow Blinking:** Standby
- **Orange Blinking:** Low Battery
- **Orange On:** Charging
- **Blue On:** Connected via Bluetooth
- **Blinking with heartbeat:** Working
- **Fast Blinking:** The device is busy

3. Left Electrode

4. Right Electrode

5. Charging Port

6. Left Base

7. Right Base

2.2 Unpacking

Main Unit; Charging Base; User Manual; USB Cable;

2.3 Symbols

Symbol	Description
	Manufacturer
	Date of manufacture
SN	Serial number
	Indicates a medical device that is not to be disposed of as unsorted municipal waste.
	Follow Instructions for Use.
	Type CF Applied Part
	MRI unsafe. Presents hazards in all MR environments as device contains strongly ferromagnetic materials.
IP22	Resistant to liquid ingress
	CE marking
	Authorized representative in the European community
	UKCA marking
	Authorized Representative in the United Kingdom
	This product complies with the rules and regulations of the Federal Communication Commission.

3. Using the Monitor

3.1 Before Use

Activation

The monitor must be charged and then activated before being used.

Note: the monitor will not work without being activated.

Charging

1. Put the device on the charging base. The Right Electrode should touch the Right Base. (Do not put in backwards).
2. Connect the charging base to your computer's USB port or a standard 5V USB charger via a USB cable.

Note:

- Ensure proper contact between the monitor and base.
- If charging fails, please clean the charging base and wipe them dry.

Configuring the Date and Time

Date and time on the monitor are automatically synchronized each time it is connected to the ViHealth app.

For connecting the ViHealth app, please refer to Chapter 3.4 Connecting to the App.

Note: To record data with the correct date and time, we recommend to connect the monitor to the ViHealth app before first use.

Standby

When the monitor is not working, it will automatically switch to standby mode.

Buzzer Beep Indicator

Beep: Starting to work

Short Beep: Heartbeat detected

Long Beep: Data saved

3.2 Connecting to the App

The ViHealth app supports to transmit data after the monitor stops measurements, and monitoring rate and EKG.

3.2.1 Downloading the App

App name: ViHealth

iOS: App Store

Android: Google Play

3.2.2 Installing the App

Install the app on an Apple product or Android device, including smart phones and tablets.

Note: For Android devices, please accept the requested permissions when you run the app.

Compatibility

The devices that are compatible with the monitor and app must meet the following minimum requirements:

iOS: iOS 9.0 or later


Android: 5.0 OS or later

Note: Android version 5.0, also known as Android Lollipop, was released in November 2014. Most devices manufactured after this date will likely have this version or a newer version installed.

3.2.3 Pairing the Monitor

1. Make sure that Bluetooth is enabled on your smart device and run the ViHealth app.
2. Tap the monitor icon in the ViHealth app, follow the connection guide on the screen to start pairing.
3. You can log into the ViHealth app after successfully pairing.

Caution: DO NOT pair the device in your smart device's settings.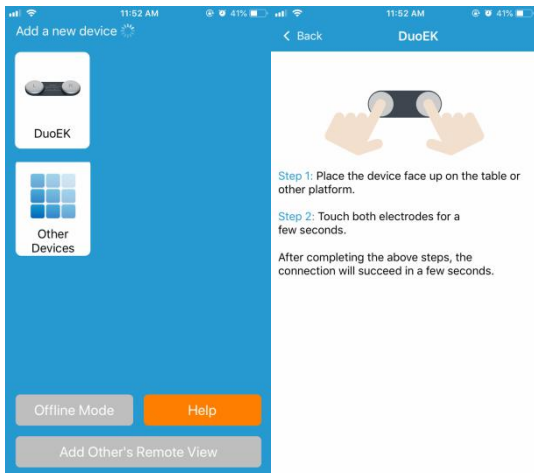

3.2.4 Transmitting Data

The monitor will automatically transmit the most recent measurement data to the ViHealth app after successfully pairing.

3.3 Taking EKG Recordings

Run the ViHealth app before measuring, you can see the real-time waveform on the app while measuring.

The monitor supports four methods for taking EKG recordings, and you can choose what suits you.

Note:


- Remain calm and still while during EKG recording.

- Remove hairs if they hamper contact with the electrodes.

A. Right Hand to Chest

Follow the instructions below to take measurements:

1. Hold the right electrode with your right hand.
2. Position the monitor across the front of your chest, just below your pectoral (chest) muscles.

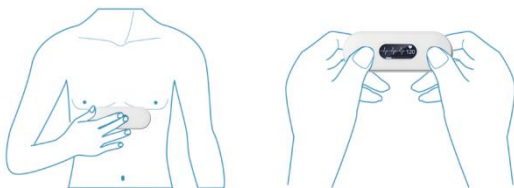3. Properly hold the electrodes onto the chest.

B. Hand to Hand

Follow the instructions below to take measurements:

1. Hold the right electrode with your right thumb.
2. Hold the left electrode with your left thumb.

C. Right Hand to Left Leg

Follow the instructions below to take measurements:

1. Hold the right electrode with your right hand.

2. Position the left electrode, so that it touches your left leg as shown.


3.4 Ending EKG Recordings

The maximum recorded time for a single EKG recording is 5 minutes. When the recording exceeds 5 minutes or the electrodes lose contact with your skin, the monitor will automatically stop measuring. If the recorded time is less than 30 seconds, the data won't be saved.

3.5 Real-time Monitoring

The ViHealth app allows you to monitor heart rate and EKG waveforms in real-time on the Dashboard screen.

The recorded data will be synced from the monitor to the app when the measurement is complete, then you can review them on History screen.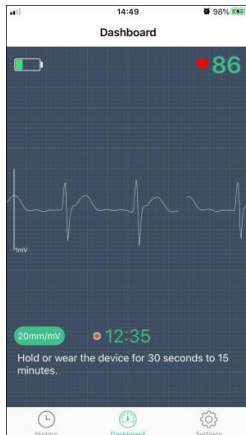

Note: Keep your monitor connected to the ViHealth app when you enable real-time monitoring.

Taking a new EKG Recording


If you want to take a new recording, you need to let the electrodes break contact with your skin, then recover them to start measuring again.

3.6 Taking Measurements without the App

The monitor supports working alone without the ViHealth app. All recording data is stored in the built-in memory of the monitor. The total recording time for all records is 20 minutes. If the total recording time exceeds 20 minutes, the oldest data will be overwritten by the most recent data.

Note: To avoid losing data, upload the data to your smart device after a measurement is complete.

3.7 Reviewing History Data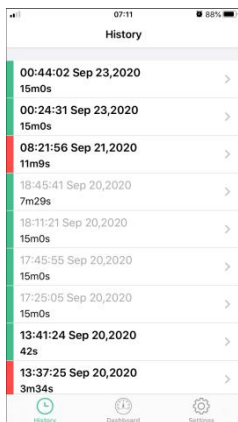

You can review history measurement results on the History screen. Tap the desired record to view more information on the EKG recording screen.

The colored bar on the left side of screen stands for the heart beat results of the current record.


-  Regular heart beat
-  Irregular heart beat

3.7.1 EKG Recording Details

Detailed items are displayed on the EKG recording screen.

If the EKG recording time is greater than 1 minute, the recording data will be divided into 1-minute segments.

The maximum number of segments is 5.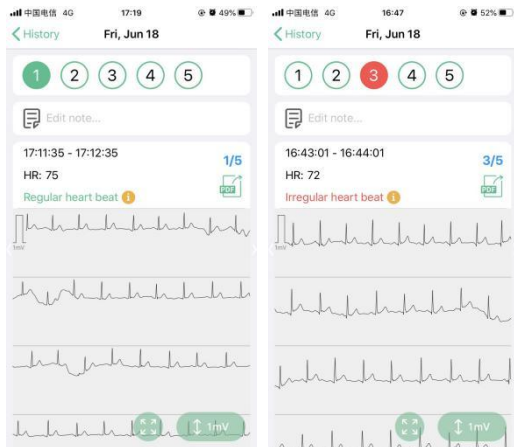

Selecting an EKG Segment

The  icon is the first segment of the EKG recording.

The green bar indicates that there is a regular heart beat in the segment.

The red bar indicates that there is an irregular heart beat in the segment.

Editing Note

Tap  to input the note about the recording.

Sharing Records

Tap  to share the current record as a PDF file.

You can choose to export the data of the selected segment or the data of entire recording.

Displaying Full Screen Waveform

Tap  to view the EKG waveform displayed full screen.

Adjusting Waveform Amplitudes

Tap  to adjust the amplitude of the EKG waveform.
Options: 0.5mV/1mV/2mV

Heart Beat Result Explanations

Regular heart beat	The recorded EKG indicates a common and regular pattern.
Irregular heart beat	The recorded EKG indicates an erratic pattern. It may be caused by an overexerted heart or other conditions.
Poor signal	Moisten skin; try another method; try to remain still during testing.

Slow heart rate	< 50 BPM
Fast heart rate	> 120 BPM

Note: Consult your doctor for more information about any health condition.


3.7.2 Deleting Records

To delete a record:

1. Use gestures to swipe left on the record.
2. Choose “Delete” to delete the current recording.

3.8 Setting the Heartbeat Sound

You can turn on/off the heartbeat sound on the monitor by choosing the [**Heartbeat Sound**] switch on the settings screen.

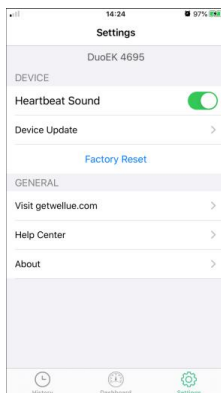3.9 Factory Reset

You can restore the device to factory settings on the monitor via the App.

Caution: All user-defined settings and measurement data will be deleted on the monitor after factory settings have been restored.

4. Maintenance

By following the directions below, you can maximize the life of your monitor.

- Do not attempt to disassemble or repair the monitor.
- Protect the monitor from sharp objects, extreme heat, or prolonged exposure to direct sunlight.
- Store the monitor in a clean, dry environment at room temperature.
- When putting on or taking off the monitor, please do not excessively twist, bend, or stretch it.
- To clean the monitor, wipe it with a soft, dry cloth or with a cloth that has been moistened in a weak solution of water and a mild, neutral detergent.

5. Troubleshooting

Problem	Possible Cause	Solution
The monitor does not connect to the App.	Your smart device is not compatible.	Check whether your smart device is compatible.
	The distance between the monitor and your smart device is too great.	Keep your smart device near the monitor.
	The monitor is connected to another device.	Check the other smart devices around you and turn off their Bluetooth.
The device cannot normally collect data.	The battery is low.	Please charge the device
	Equipment damage.	Please contact your local agent
Abnormal heart rate	Poor contact between the skin and the monitor electrodes.	Moisten the electrode areas of the monitor.
EKG waveform is disordered, and greatly cluttered.	Measurement method is incorrect.	Refer the manual and ensure you're wearing it correctly.
	Poor contacts for EKG electrodes.	Please clean the EKG electrodes according to the method described in the manual.
EKG waveform drifts.	Hand or body may be moving.	Try to keep your body perfectly still.

6. Specifications

Size	92×32×8.2 mm (main unit)
Weight	26.8g (main unit)
Wireless connectivity	Bluetooth 4.0
Connector	Micro-USB connector
Battery type	Rechargeable lithium-polymer battery
Battery run time	>500 checks
Charge time	2 -3 hours
Heart rate range	30 to 250 BPM
Device built-in storage	10 sessions, if EKG length is less than 1 minute; 4 session if EKG length is 5 minutes
In-App storage	Unlimited
EKG length	30 seconds to 5 minutes
App for iOS	iOS 9.0 or above, iPhone 4s/iPad 3 or above
App for Android	Android 5.0 or above, with Bluetooth 4.0 BLE

FCC Statement

FCC Warning:

FCC ID: 2AD XK-3622

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with part 15 of the FCC Rules.

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

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Shenzhen Viatom Technology Co., Ltd.
Prolinx GmbH
Prolinx Global LTD