

INSTRUCTION MANUAL

DIGITAL BLOOD PRESSURE MONITOR

Model Prolife PX7 Premium (BP168A)*

1. Introduction	24
1.1 Safety instructions	24
1.2 Intended use	24
1.3 Intended users	24
1.4 Intended patient population	24
1.5 Intended use environment	24
1.6 Intended use accessories	24
1.7 Contraindications	24
1.8 Intended clinical benefits to patients	24
1.9 Introduction to the working principle	24
2. Important safety information	24
2.1 Warning	24
2.2 Caution	24
2.3 General precautions	24
3. Know your device	24
3.1 Operating key	24
3.2 Digital LCD display description	24
3.3 Common functions	24
3.4 Functional Description	24
3.5 Preparation for a measurement	24
4. Preparation before use	24
4.1 Installing batteries	24
4.2 Setting date and time	24
5. Use equipment	24
5.1 Applying the arm cuff	24
5.2 Sitting correctly	24
5.3 Taking a measurement	24
5.4 Discontinuing a measurement	24
5.5 Using memory functions	24
5.6 Bluetooth function	24
6. Useful information	24

*Need for the user to follow this instruction manual thoroughly for your safety. Please keep for future reference. For specific information about your own blood pressure, consult your physician.

*In the text of the instruction manual, abbreviated name of device is used: Digital Blood Pressure Monitor Prolife PX7 Premium.


3.3 Common Functions:


- Blood pressure and heart rate are measured.
- Memory storage and clearing functions.
- Date and time Settings.


▲ Note: The above basic normal functions can be safely used by the user

3.4 Functional Description

Low battery warning

If the early battery warning  icon appears in the display, the batteries remain 20% power to warn user the batteries will run out soon.

If the battery warning  icon appears in the display, the batteries are empty and must be replaced.

▲ Attention! After the battery warning  icon appears, the device is blocked until the batteries have been replaced.

3.5 Preparing for a Measurement

